

Face Shield

"Hounisen"

Single-use protection
against viral infection

Instruction for use



Tested and approved
by Medical Departments of
Infection in Denmark



With glasses:

Turn the round nubs in towards the forehead. This provides optimal fit for glasses.

Without glasses:

Turn the round nubs away from forehead.

One size.

The top edge is placed on the back of the head with the shield down in front of the face.



The shield now sits as shown, protecting the eyes, nose and mouth against splashes from coughing and sneezing, while also preventing the wearer from touching the face.

Facts:

This is a disposable face shield and must be discarded after use.

The shield is manufactured from 100% recyclable PET plastic

The shield is approved as PPE by the Danish Safety Technology Authority.

Manufactured by HN Group A/S in Billund.
ISO 13485 medical device certification.

