




Step 1



Step 2



Step 3

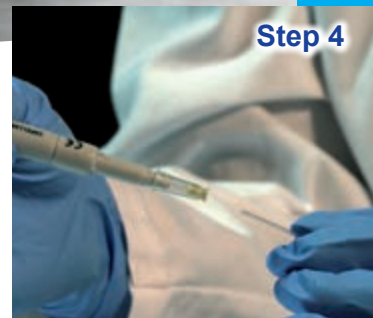

Step 4

Vitrex® Capillary Dispenser

Vitrex Capillary Dispenser is a unique liquid handling tool, developed to empty capillary tubes for blood or other liquids.

FULLY AUTOCLAVABLE

Autoclavable at 121°C & 15 psi for sensitive applications.

LOW COST

This dispenser is specially designed for use with Diagnostic Kits.

UNIQUE TIP CONE

Designed to accept most Vitrex Capillary Tubes.

OPTIMUM SIZE

At a length of 130 mm this dispenser is the optimum practical size for blood collection.

Technical Specifications

| Colour Code | Vol. µl | Pcs. | Article No. |
|-------------|-----------------------------|------|-------------|
| Grey | Most Vitrex Capillary Tubes | 1 | 569013 |

Recommended use

Step 1: Loading the device

Insert an unused capillary tube into the pipette tip. The end of the capillary tube must be visible in the tip.

Step 2: Filling the capillary tube

Hold the pipette horizontally and let the capillary tube be filled by capillary effect. Be sure not to press the plunger of the pipette.

Step 3: Emptying the capillary tube

Empty the capillary tube by pressing the plunger gently to the first stage. The liquid will be dispensed, but a small amount of liquid will be left in the tube. After a pause continue to depress the plunger to the second stage. The small amount of liquid left in the tube has also dispensed.

Step 4: After use

Remove the capillary tube carefully and discard in a suitable safety container.